

menox

Mini Stamp Pedal extension


A.321 and 322

Manual




Contents: Section	Subject	Page
1	Table of Content	2
2	Introduction	2
3	General Information	2
4	Instructions for use	2
5	Installation picture	3
6	Installation Instruction	3
7	Warranty	4
8	Maintenance	4
9	Certification	4

Art. no 321 for Automatic gearbox

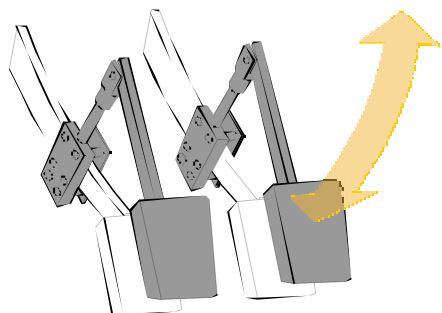

Art. no 322 for Standard gearbox

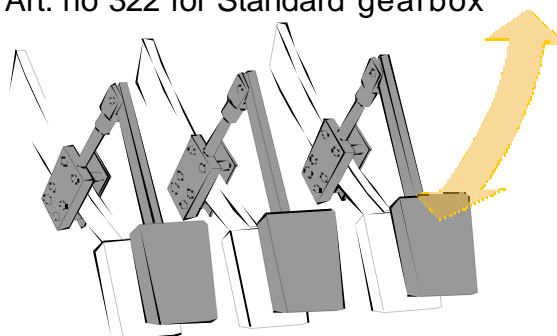

Congratulations! You have purchased a Menox Mini-Stamp Pedal Extension. We wish you pleasant and safe driving

2 Introduction

This instruction manual must be read and its instructions understood before installing and using the Menox Mini-stamp. Keep this manual for later use. If the manual is not correctly filled in, the manufacturer is released of liability. The manufacturer is not responsible for incorrect installation. For safety reasons, only Turvax Oy or a trained and authorized installer may install and service this product.

3 General Information

The Menox Mini-stamp is a driving aid developed for disabled drivers who need a short pedalextension of up to 10cm. Menox is a safe and tested product. It can be installed into all vehicles and adjusted according to the driver's needs.

The Mini-stamp is equipped so it can be easily turned up out of the way giving other drivers quick and easy access to the vehicle.

The Menox Mini-stamp is universal and can be installed into most makes and models.

4. Instructions for use

The Mini-stamp is very simple to use. Pull the pedal into place and ensure that the back bolt rests upon the surface of the OEM pedal. Simply swing the pedal up to clear the pedals for original driving.


When not in use, the Mini-stamp must be lifted up out of the way and ensured that it is firm in place.






Ensure that when you lift the pedal up, it does not come in contact or catchhold on the surroundings in the drivers side floorpace.

When the pedal is lifted up, also ensure that the extensionbolt off the back of the Mini-stamp will not come into contact with the other drivers foot.

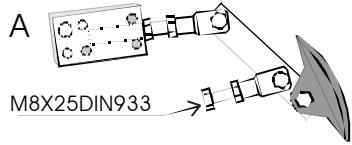
5. Installation pictures

Contents

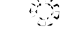


- C**  8-5/16" DIN741, 2 pcs
- D**  M8X30DIN933, 1 pcs
- E**  M8X40DIN933, 5 pcs
- F**  M8X60DIN933, 1 pcs
- G**  DIN6798-A 4 pcs


Brake- and clutch pedal installations

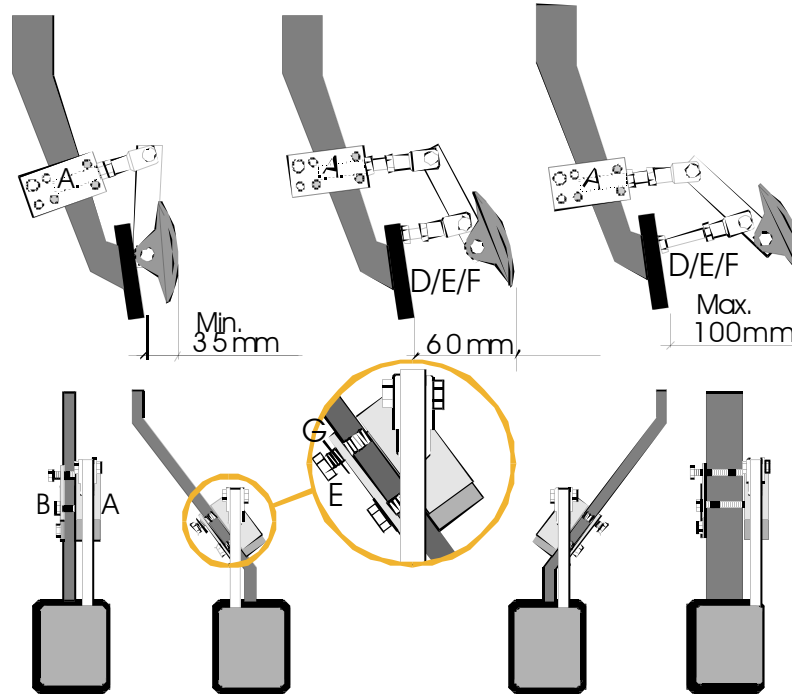


Down

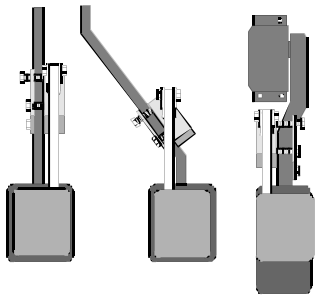

Up

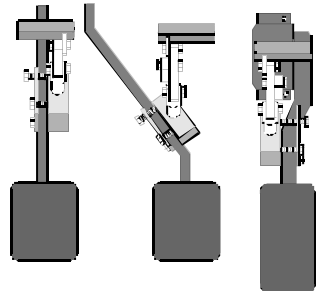

Gas pedal installations

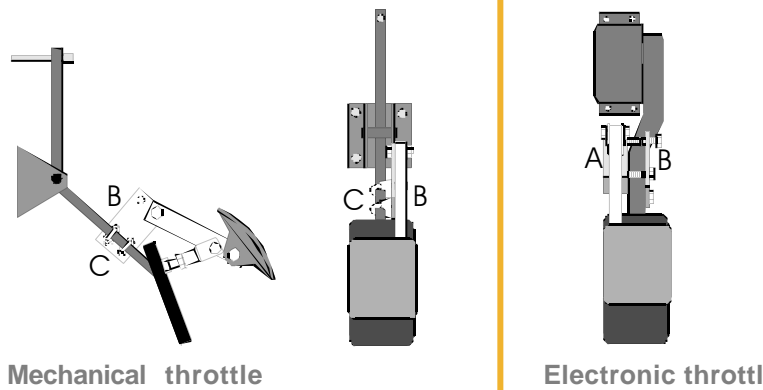

6. Installation sequence.

In the installation package you will find a variety of lengths of bolts. First determine the length of extension needed. Once you have determined the length of extension you can fasten it to the pedal of the vehicle. The bolt on the fork joint on the backside of the Mini-stamp pedal can be adjusted to the right length. After the right length has been found tighten the nut so that length of the bolt is secured. The longer the extension, the lower the Mini-stamp will be located on the OEM pedal.

Open the three bolts on part A of the Mini-stamp and place the plate near the OEM Brake and or clutch pedal. Place B on the other side of the pedal and tighten. Note plate B will bend and this is normal to allow for the plate to find the tightest position.

To install the gaspedal, follow the gaspedal installation figure. Note that by removing plate A, and using plate B and part C you can make a lighter connection to the accelerator.

Now that the pedal is attached to the other OEM pedals, you can begin the final fastening of the bolts and nuts. Tighten the hinge part of the Mini-stamp. The hanging part is equipped with two springwashers. Tighten these so that the movement is firm and that the pedal securely holds it self in the upright position. Also make sure that when the pedal is lifted up that it does not come into contact with the above plastic. Press the brake, clutch and the accelerator to ensure that when the pedal is in an upright position it will not catch on anything and possibly cause it to jam.

Km / mile	Date	Service Station and mechanic

7 Menox warranty

7.1 Duration and terms of warranty

7.1.1 Duration of warranty

a) We guarantee a warranty of two (2) years for the product.

7.1.2 Extent

Our warranty covers all faults occurring during manufacturing and materials.

7.2 Terms

The warranty is valid provided that:

- a) Installation is performed by an authorized mechanic.
- b) A warranty repair be negotiated by the manufacturer/dealer of the product prior to action of repair
- c) The use, maintenance and physical conditions in which the product has been used have been consistently normal and the operation instructions have been followed and scheduled servicing has been performed.

7.3 Proper actions in warranty cases

In warranty cases, always contact a garage or mechanic authorized to perform the warranty repair.

7.3.1 The defective product is returned to the manufacturer and after they have examined it, the repaired or new product is sent to the customer.

7.3.2 In the case that there is need for compensation for work, it must be negotiated with the manufacturer. Compensation can amount to a maximum of one(1) hour at the regular rate of the garage/mechanic. Compensation is paid as a reduction from the purchase bill/receipt.

7.3.3 The warranty does not cover in direct costs such as shipping or travel expenses or malfunctions caused by normal wear.

8 Maintenance Guide

All mechanical devices should be serviced and that includes The Mini-stamp. The purpose of scheduled maintenance is to prevent the product from malfunctioning while driving. Servicing takes place in connection with a car's scheduled maintenance. It can be done by a Menoxinstaller, an automobile mechanic or anyone with sufficient expertise.

8.1 Mini-Stamp Inspection.

Check the overall function of the system. Check for any unnecessary looseness. Make sure that the Mini-stamp pedal moves up and down and does not come into contact with anything. Also ensure that the Mini-stamp stays securely upright when not in use. If anything seems out of place or peculiar please contact the manufacturer for further advice.

9 Manufacturer's Certificate

I certify that the Menox Mini-Stamp has been designed and manufactured in Turvax Oy, Kempele Finland. Throughout the design and manufacturing, special attention has been placed on safety and ease of use of the product. Menox Mini-Stamp's are manufactured in accordance with Finnish guidelines (703/98) 103§, ISO/TC22/WG4 and EU directives (74/60/ETY,78/632/ETY).

Respectfully yours,

Juha Paananen

CEO



Left Foot Accelerator
Serial Nr.Sticker

Manufacturer



Tarjusojantie9
FIN-90450 KEMPELE
FINLAND
Tel.+3588513624
Fax+35885563803
e-mail: menox@menox.org



For successful driving

[www . menox . org](http://www.menox.org)